

TrioMed Active Surgical and Medical Mask

INTENDED PURPOSE:

The main purpose of this surgical mask is to prevent the wearer from spreading contamination. The TrioMed Active Surgical & Medical Mask is a single use disposable facemask intended to protect the patient from contamination due to microorganisms, bodily fluids, and air borne particulate material.

INSTRUCTIONS:

Product can be worn continuously for an 8-hour work shift.

Product has a shelf life of 5 years.

For external use only.

Store at room temperature away from direct sunlight.

Dispose of according to your country's regulations.

CONTRAINDICATIONS:

Discontinue use and consult a physician if irritation or sensitivity develops.

This product contains iodine, it should not be used by persons with known iodine hypersensitivity.

WARNINGS:

The product helps protect against certain particulate contaminants but does not eliminate exposure to or risk of contracting the disease or infection.

Any serious incident that occurs when using this device should be reported to the manufacturer and the national competent authority.

PRODUCT BENEFITS:

Non-cytotoxic, latex free and non-flammable

The outer active layer with TrioMed antimicrobial technology (iodine based) is to protect the external surface from microbial deterioration. The iodine is embedded in the external surface therefore does not leach or escape the material, its purpose is to render bacteria and viruses inactive when they come into contact.

HOW TO PUT ON THE MASK

- Remove a mask from the box or pouch and make sure there are no obvious tears or holes in either side of the mask.
- Determine which side of the mask is the top. The side of the mask that has a stiff bendable edge is the top and is meant to mold to the shape of your nose.
- Determine which side of the mask is the front. The front side is with the embossed marking "OUT". The colored side of the mask is usually the front and should face away from you, while the white side touches your face.
- Hold the mask by the ear loops. Place a loop around each ear.
- Mold or pinch the stiff edge to the shape of your nose.
- Pull the bottom of the mask over your mouth and chin.

Explanation of symbols:



Indicates the item is a medical device



Obelis S.A. Bd. Général Wahis 53
1030 Brussels, Belgium
Tel: +(32) 2 732-59-54
Fax: +(32) 2 732-60-03
mail@obelis.net



I3 BioMedical Inc.
14163 boul. Curé Labelle Suite 50
Mirabel, Québec, Canada J7J 1M3
WWW.I3BIOMEDICAL.COM